

SculpSure®

Pre-Treatment | 治疗前

Treat To Complete: Set Realistic Expectations
治疗至完成：设置实际期望

- # of applicators needed
所需的敷料器数量。
- # of treatments needed to achieve desired results
达到目标结果所需的治疗数。

Encourage proper hydration and food intake prior to treatment
在治疗前，鼓励摄取适量的水分和食物

Enter the appropriate demographic information:
输入适当的人口统计信息：

New: This client has never been to your practice.
新客户：您未曾诊疗过此客户。

Current: This client has previously had services at your practice.
现有客户：此客户先前曾在您的诊疗下使用您的服务。

Returning: This client has previously had a SculpSure treatment at your practice.
回流客户：此客户先前曾在您的诊疗下接受 SculpSure 治疗。

CYNOSURE®

Palomar
ConBio
ellman

sculpsure.com.au



SculpSure®

Sculpsure Treatment | SculpSure 治疗

Position and Place | 准备就定位

Place the belt on the bed, position patient, and attach the frames.
将带子放在床上，让患者就位，并系上框架。

Tighten belts so frames are positioned firmly against the skin and there is a pillowing of tissue within frame.
系紧带子，让框架固定于皮肤，在框架内有枕垫。

Apply Lux Lotion | 涂上 Lux 乳液

Apply only a light sheen of Lux Lotion to the skin where the contact sensors sit.

在置有接触传感器的皮肤涂上薄薄一层 Lux 乳液。

Note: Using excessive Lux Lotion can result in error message (codes F508-F511) which will be displayed on the user screen if applicators are not dry before use. | 请注意：在使用敷料器未经乾燥的情况下，若过度使用 Lux 乳液，将造成用户端屏幕上显示错误信息（代码 F508-F511）。

When securing the applicators in the frame, listen for two clicks.

在确保框架内的敷料器时，聆听两个喀哒声。

This ensures they are securely seated.

这能够确保其稳当的放置。

Build Phase: 4 minutes | **Sustain Phase:** 21 minutes

建立阶段： 4 分钟 | **持续阶段：** 21 分钟

CYNOSURE®

Palomar
ConBio
ellman

sculpsure.com.au



SculpSure®

Clinical Endpoint | 临床终点评估

Clinical Endpoint is the Zone Score 4 –

NOT the energy setting

临床终点评估为区域分数 4 – 非能量设置

- Below 4 – Increase energy | 低于 4 – 增加能量
- Beyond 4 – Advance to Cool (ATC) | 低于 4 – 提前冷却 (ATC)

Client Comfort | 客户舒适度

If the client reports Zone Score >4 over entire treatment area, press ATC. Once cooling is complete, adjust the energy back to the pre-ATC setting.
若客户反应整体治疗区域的区域分数大于 4，则按下 ATC 按钮。一旦冷却完成，将能量调回 ATC 前的设置。

If after adjustment the client still reports too much intensity, press ATC again and DO NOT readjust the setting back to pre-ATC settings.
若调整后，客户仍反应太强烈，则再次按下 ATC 按钮，切勿重新调整回 ATC 前的设置。

ATC in Build Phase | 建立阶段的 ATC

- 40 seconds cooling | 冷却 40 秒

ATC in Sustain Phase | 持续阶段的 ATC

- 20 seconds cooling | 冷却 20 秒

Loss of contact >30 seconds | 失去接触超过 30 秒

- Automatic 4.5 minutes of cooling will begin.
将自动开始冷却 4.5 分钟。

Clinician MUST hit the start button to resume treatment.
临床医师必须按下开始按钮才可恢复治疗。

Dealing with hot spots | 处理热点

Confirm long axis of applicators are in full contact with skin, frames are positioned firmly against skin, and applicators are 90° to treatment area. Press and hold the applicator graphic in question until the screen flashes a pale blue. Leave the applicator in this suspended mode for 5-10 seconds. | 确认敷料器的长轴完全与皮肤接触、框架牢固地与皮肤接触、敷料器与治疗部位呈 90 度。按下向内的敷料器图像，直到屏幕闪烁着淡蓝色。让敷料器处于暂停模式 5 到 10 秒。

CYNOSURE®

Palomar
ConBio
ellman

sculpsure.com.au

